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INDICAZIONI

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900 6209

CYBERTECH
TEMP LONG
NON EUGENOL

Resin cement for long term & temporary implants
• INSTRUCTIONS FOR USE

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INSTRUCTIONS FOR USE

INDICATIONS FOR USE

CyberTemp Long is a non-eugenol, acrylic-urethane polymer based temporary cement for implant luting. Cementation of permanent implant - retained crowns.

DIRECTIONS FOR USE

1. Check fit, occlusion and proximal contacts. Complete all finishing procedures.
2. Dry and isolate the prepared abutments. Absolute dryness is not necessary, but obvious contamination should be avoided.
3. Make sure the interior of the restoration is clean and dry. To facilitate future removal of implant restorations, lubricate the surface of the abutment with a thin layer of a water-based lubricant.
4. Bleed the syringe before installation of the automix dispensing tip. Mount a new, automix dispensing tip onto the CyberTemp Long dual-barrel syringe and bleed the syringe again. This insures complete flow of base and catalyst components. Syringe a thin layer of the cement directly into the restoration. You have 45-60 seconds working time.
5. Firmly seat the crown and either hold in place or have the patient gently close on a cotton roll until the initial set which occurs in 2-2.5 minutes. The initial set is characterized by a distinct gel phase. Excess cement can be removed safely after 2.5 minutes, but before final set.
6. Final cement set is between 4-5 minutes, depending on conditions.

HAND-MIXING DIRECTIONS

1. Extrude the appropriate amount of base and catalyst onto the selected mixing surface.
2. Using a clean spatula or mixing stick, mix the two components into a homogenous paste. Using the mixing stick or brush, apply a thin layer of the mixed cement on to the cleaned, inside surface of the restoration.
3. You have approximately 45 seconds of working time from start of mix.
4. Firmly seat the restoration in place. Initial gelsset occurs within 2 to 2.5 minutes after seating. Cleaning of excess cement should be initiated immediately after this gel-set phase.
5. Final cement set occurs between 4 and 5 minutes after seating of the restoration.

PRECAUTIONS

Though rare, some individuals may be sensitive to this product. If an allergic reaction occurs, discontinue use, remove from patient and refer patient to a physician. Important Application Information: Only a small bead of CyberTemp Long around the margin is necessary to securely hold your crown. This is especially true with highly retentive restorations. Using less cement saves money and allows for easy retrievals!

ESSENTIAL INGREDIENTS

Base & Catalyst:

- Urethan dimethacrylate.
- Triethyleneglycoldimethacrylate.
- Polymerization activators.
- Fumed Silica.

STORAGE

- Do not store above 24°C (75°F).
- For optimum freshness, keep refrigerated.
- Do not freeze.
- If refrigerated, allow the syringe to reach room temperature.
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.

According to the EU Medical Devices Regulation, users / patients are obliged to report serious events with a medical device to the manufacturer and to the competent authority of the country in which they occurred.

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