



Instruction for Use

Gutta Percha Points

Hand-rolled endodontic points used to fill a prepared root canal prior to tooth restoration.

Composition

Gutta percha 22 ~ 27%, Zinc oxide 60 ~ 65%,
Barium sulfate 8 ~ 13%, Coloring agent 0.2 ~ 0.5%

Application

Preparing the root canal

- Opening the tooth
- Shaping the canals

Filling the canals with Gutta Percha Points matched to the file used.

Precaution

- Professional use only.
- Do not autoclave or heat sterilize
- Store in a cool area and avoid exposure to sunlight
- Single-use device / Re-use is not allowed.

Shelf Life

- 4 years from the date of manufacture

Symbols EN ISO 15223-1:2016



Use by Date



Lot number



Manufacturer



Single Use



Medical Device

Warranty

CB Healthcare Consulting GmbH will replace the product that is proven to be defective.

CB Healthcare Consulting GmbH does not accept liability for any damage or loss, direct or consequential, stemming from the use of or inability to use the product described. Before using, it is the responsibility of the user to determine the suitability of the product for its intended use.

The user assumes all risk and liability in connection therewith.