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EDTA

97-92289

Gel for root canal cleaning and preparation

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DE Healthcare Products
DEDICATED TO EXCELLENCE

CB Healthcare Consulting GmbH

(manufacturer)

Am Neumarkt 34

22041 Hamburg, Germany

REG. 200207

EDTA Gel for root canal cleaning and preparation

Applications

- root canal preparation with files and reamers.
- presence of calcification in the canal.
- too narrow canals in posterior teeth.

Mode of action

- removal of inorganic substances by chelation (EDTA-CA2+).
- easier filling due to its lubricating action.
- produces the cleanest canal wall after smear layer removal.

Directions

1. Place a small amount in the canal orifice using the syringe
2. Carefully push into the canal with a fine file or reamer
3. Re-apply EDTA when necessary or until cleaning and shaping is complete
4. Rinse off EDTA by thorough irrigation with sodium hypochlorite.

Rotary Ring

Putting a syringe into a rotary ring is more convenient to handle.
Use the rotary ring as a wrench to tighten or loosen a tip.

Cautions

Use EDTA with discretion to prevent perforation.
Avoid extrusion of the solution past the apex.
Keep the syringe capped when not in use.

Expiry Date

2 years from production date.

Warning: Causes serious eye irritation. IF IN EYES: Rinse cautiously with water for several minutes.

Remove contact lenses, if present and easy to do. Continue rinsing **Contains:** Ethylenediaminetetraacetate (EDTA). 19%

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