Before using this product, please carefully read and follow the recommendations of the instruction manuals originally delivered with your device and system. Please pay special attention to the safety precautions.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

Intended use

The Air Polishing powder, Perio is intended to be used with EMS AIRFLOW® devices equipped with EMS AIRFLOW® and/or PERIOFLOW® Handpiece to remove biofilm supragingivally and subgingivally:

- From primary and secondary teeth.
- From dental hard tissues: enamel, dentine and cementum.
- From oral soft tissues: gingiva, tongue and palate.
- From restorations, prosthetics and dentures.
- From implants.
- From orthodontic appliances including aligners.

Indications

This powder is indicated for air polishing: Removes biofilm supragingivally and subgingivally in >4 to 9mm periodontal, root furcations and peri-implant pockets.

This powder prepares for:

- Caries diagnosis.
- Fluoride treatment.
- Orthodontic treatment.
- Restorative and prosthetic treatments.
- Bleaching procedures.

Intended user population

This powder is intended to be used by dental professionals.

Intended patient population

This powder is intended to be used on patients regardless of age, including children.

Undesirable side effects and contraindications

Glycine can cause allergic reactions. This powder should not be used on patients allergic to Glycine.

Recommendations

Always disclose biofilm. Do not overtrear. Stop the treatment when/ where the disclosing agent is gone.

Storage and shelf-life

Storage condition up to 25°C and up to 90% relative humidity. Transport conditions between -29°C to 38°C and up to 90% relative humidity. Shelf life: 26 months.

Disposal

There is no specific safety risk related to the disposal of the product and its container. Disposal must be made according to country specific regulations.

Main ingredient

Glycine

Contains

Hydrophobic modified amorphous silica

Translations in this manual are based on the English version. Thus the English text will be referred to in case of discrepancy.

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Dr. Wittmann GmbH & Co KG
B.A. INTERNATIONAL LTD.