125,00 mm

AIR POLISHING POWDER GLYCINE

• Soft

97-96398

Polvo para aeropulidor

Poudre abrasive

Prophylaxepulver

Polvere abrasiva

Pó para Polimento

200g e

CB Healthcare Consulting GmbH

(manufacturer)

Am Neumarkt 34
22041 Hamburg, Germany

MADE IN GERMANY

Rev.2018/ 10

DE HT™ DE Healthcare Products
DEDICATED TO EXCELLENCE

Shake well before use

LOT

75,00 mm

S1

125,00 mm

(EN)

Type of application:

Glycine-based prophylactic powder for the careful removal of supragingival deposits and discolorations. Suitable for all trade-standard air polishing systems.

Instructions on use:

Please pay attention to the maximum filling height of the device manufacturer. Not suitable for the removal of deposits or concretions in gingival pockets. Furthermore, do not direct the powder stream on the gums or other soft tissue. Exposed dentine and enamel caries should not be treated with the air polishing system, as higher abrasive values can be expected. Composite, compomer, amalgam and gold fillings are also not allowed to be treated, as doing so might roughen the surface.

Patients who suffer from chronic bronchitis or asthma are not allowed to be treated with an air polishing system and material. Do not use if sensitive to glycine. Patients and operators have to wear safety goggles. The operator must wear a mask and gloves. Use an evacuator. Apply Vaseline to the patient's lips. Only for dental use. Keep the containers closed. **Shake well before use!**

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