125,00 mm

AIR POLISHING POWDER TAGATOSE

- Perio plus supragingival & subgingival

97-96399

Polvo para aeropulidor

Poudre abrasive

Prophylaxepulver

Polvere abrasiva

Pó para Polimento

100g e

DE Healthcare Products
DEDICATED TO EXCELLENCE

Shake well before use

 **CB Healthcare Consulting GmbH**
(manufacturer)
CE Am Neumarkt 34
22041 Hamburg, Germany
MADE IN GERMANY
Rev.2018/10

LOT

S1

(EN)

Type of application:

Natural sugar-based prophylactic powder (non-cariogenic) for the careful removal of supragingival and subgingival deposits and discolorations. Suitable for all standard air polishing systems.

Instructions on use:

Please observe the maximum filling height of the device manufacturer.

Subgingival application:

• Removal of periodontal biofilm • Cleaning of implants • Further treatment of periodontal diseases after completion of initial treatment

Supragingival application:

• Removal of supragingival biofilm and discolorations around the sulcus • Removal of discolorations in sensitive areas • Polishing of the teeth, final polishing is not necessary • Suitable for regular prophylactic treatment

Patients who suffer from chronic bronchitis or asthma, are not allowed to be treated with an air polishing system. Patients and operators must wear safety goggles. The operator must wear a mask and gloves. Use a high-performance suction device. Apply Vaseline to the patient's lips. The powder stream should not be directed at fillings, crowns and bridges as these dental restorations could be damaged. Only for dental use. Keep the containers closed.

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