SAFE-ST EVAC DISINFECTION CONCENTRATE

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Danger/Peligro/Danger/Gefahr/
Pericolo/Gevaar/Perigo

 **CB Healthcare Consulting GmbH**
(manufacturer)

Am Neumarkt 34
22041 Hamburg, Germany

Tel: +49 40 656680

Email: info@cbhealthcare-intl.com

Rev 2020/01

 **DE Healthcare Products**
DEDICATED TO EXCELLENCE

5°C
41°F

40°C
104°F

 **0482**

EN: Areas of application: High performance solution designed for cleaning and disinfecting of dental Evacuation systems (incl. suction hoses and amalgam separator) and spittoons. **Instructions for use:** The alcohol-free concentrate must be first diluted with water. Recommended application concentration: 2%. Mix 20 ml of concentrate with 980 ml of tap water and aspirate the solution through the suction hose. Regularly, the suction system should be disinfected once or twice a day. The spittoon should be disinfected by pouring approx. 250 ml of the ready-to-use solution. Do not mix with other disinfectants or cleansers! Always prepare ready-to-use solutions fresh and use immediately. **Effectivity:** Bacteria and Candida albicans in accordance to VAH (high contamination); 2.0 % - 5 min.; 1.0 % - 60 min.; Bacteria in accordance to EN 13727 and EN 14561 (high contamination); 3.0 % - 2 min.; Yeastocidal (Candida albicans) in accordance to EN 13824 and EN 14562 (high contamination); 3.0 % - 2 min.; Limited virucidal (Enveloped viruses i.e. HBV, HIV, HCV) in accordance to GGA (RKI)/DVG (contamination); 2.0 % - 2 min. **Virucidal (non-enveloped viruses):** Causes severe skin burns and eye damage. Toxic to aquatic life with long lasting effects. Avoid release to the environment. Wear protective gloves/ eye protection. IF ON SKIN: Wash with plenty of water. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If (skin irritation occurs): Get medical advice / attention. If eye irritation persists: Get medical advice / attention. **Disposal of contents/ container in accordance with all local, regional, national and international regulations.** Contains N,N-Dimethyl-N-Di-octylammoniumchlorid, potassium hydroxide. **Directive 648/ 2004/ CE:** Contain < 5% nonionic surfactants, < 5% cationic surfactants, fragrances.

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