NANO SPITTOON FOAM CLEANER/ DISINFECTANT

97-98387

Area of application: The Nano Spittoon Foam is a cleaning, disinfecting and maintaining spittoons in Dental treatment units. It steadily removes water stains, blood and colour residues of plaque tablets with its active cleaning system. Not suitable for surfaces sensitive to alcohol. **Indications for use:** Shake before use and apply undiluted. Press and hold the button, pointing the nozzle over the sink and completing cover the basin. After 1 minute of contact: rinse with water and clean down by hand if necessary. **Microbiological Effectiveness:** Bactericidal in accordance to EN 13697, levurocidal in accordance to EN 13697 (Candida albicans), tuberculocidal according to EN 14348. **Contact times:** Bacteria and Candida albicans (EN 13697): 1 min. (low bio load) Tb (Tuberculosis) in accordance to EN 14348: 5 min. (low bio load) **Safety instructions:** Pressurized container. Keep out of the reach of children. H222 - Extremely flammable aerosol. Keep away from heat/sparks/open flames/hot surfaces. - No smoking. Do not spray on an open flame or other ignition source. Pressurized container: Do not pierce or burn, even after use. Avoid breathing dust/fume/gas/mist/vapours/spray. Protect from sunlight. Do not expose to temperatures exceeding 50°C/ 122°F.

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400 mLe

DE Healthcare Products
DEDICATED TO EXCELLENCE

Danger
Peligro
Danger
Gefahr
Pericolo
Gevaar

see imprint
ver la fecha impresa
voir impression
siehe Aufdruck
vedi iscrizione
zie opdruk

520 3

(manufacturer)

CB Healthcare Intl. s.r.l.
Via Partisani, 3
47016 Fiumana, Predappio (FC), Italy
Tel: +39 0543 929 111
Rev. 2016/08