↓ Dobra 6

HENRY SCHEIN®
INTERDENTAL WOODEN WEDGES

ES Cunas interdentales de madera
FR Cales interdentaires en bois
DE Interdentalkeile aus Holz
IT Cunei interdentali in legno
NL Interdentale houten wiggen
PT Cunhas interdentais de madeira
CS Mezižubní dřevěné klinky
PL Drewniane kliny międzyzębowe

⑩ Interdental wooden wedges
 Henry Schein interdental wooden wedges are indicated for general use in restorative procedures, with the mechanical function of adapting matrices and/or separating adjacent teeth. The interdental wooden wedges do not require cuts or adjustments, because its shape allows an easy insertion into the interproximal areas and ensures a perfect adaptation of the matrices. **Usage:** Select the most appropriate wedge size for the procedure. Insert the wedge into the interproximal space. Once the procedure is completed, the wedge must be removed. **Warnings, precautions and restrictions:** Professional use only. Single use only. Its reuse may cause cross contamination. If there is proven allergy to any of its components, the product must not be used. To avoid an accidental swallowing of this product, it is necessary the application of a total isolation technique of the operatory field. The product has a sharp point and must be handled with caution to avoid risk of causing harm to the patient. **Composition:** Natural Wood. Colored: Wood and non-toxic reactive dyes. **Storage:** To be stored in a cool and dry place, protected from sunlight.

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