# Maxima® by HENRY SCHEIN®

900-6281

## Non-Alcoholic Surface Disinfection

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**Indications for use:** Use on the foam setting. Spray the ready to use solution as foam on surfaces from a distance of 20-30 cm. With the use of a lint-free coloursafe cloth, thoroughly wipe over the surface and leave to dry. Excess foam to be removed with a disposable cloth. for use on larger surface areas, made of sensitive material such as orthiered leather, test it first on a smaller area for compatibility. To avoid streaks from forming, a regular wiping with a moist (water) disposable cloth is recommended. **Microbiological Effectiveness/Effectivity:** Bacteria (incl. MRSA) in accordance to VAH/DGHM, EN 1040, EN 1276, EN 13697 (high bio load); Conc.: 1min. *Candida albicans* in accordance to VAH/DGHM, EN 1275, EN 1650, EN 13697 (high bio load); Conc.: 1min. membrane covered viruses (i.e. HBV, HIV, HCV, H1N1, H5N1) in accordance to BGA (RKI)/DVV (with bio load); Conc.: 1min. (with bio load). Rota-viruses in accordance to BGW/DVV; Conc.: 1min. Noro-viruses in accordance to EN 14476 (clean); Conc.: 30min. **100g contains:** 0,24g Dialkyldimethylammoniumchlorid 0,48g Alkyl(Aryl)dimethylammoniumchlorid

DISTRIBUTED BY:  
DISTRIBUIDO POR:  
DISTRIBUÉ PAR:  
DISTRIBUITO DA:  
**HENRY SCHEIN INC.**  
MELVILLE, NY 11747 USA  
**[EC REP]**  
HENRY SCHEIN U.K. HOLDINGS LTD.  
GILLINGHAM ME8 0SB U.K.  
WWW.HENRYSCHIEINBRAND.COM

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