SAFE R EVAC SYSTEM LIQUID

97-92532

Evacuation System Disinfection Concentrate
Concentrado de desinfección del sistema de evacuación
Désinfectant d'unités d'aspiration Concentré
Absauganlagen Desinfektion Konzentrat
Concentrato disinfettante per impianti di aspirazione
Concentrado de desinfeção do sistema de evacuação

High performance solution designed for cleaning and disinfecting of dental Evacuation systems (incl. suction hoses and amalgam separator) and spittoons.

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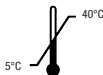
EN: Indications for use: The concentrate must be first diluted with water. Recommended application concentration: 2%. Add 20 ml concentrate with tap water to 1.000 ml and aspirate the solution through the suction hoses. Regularly, the suction system should be disinfected once or twice a day. The spittoon is disinfected by purging approx. 250 ml of the ready-to-use dilution. Do not mix with other disinfectants or cleansers! Always prepare ready-to-use solutions fresh and use immediately. Effectivity: Bacteria and Candida albicans in accordance to VAH (high contamination): 2.0% 5min, 1.0% 60min. Bacteria in accordance to EN 13727 and EN 14561 (high contamination): 3.0% 2min. Candida albicans in accordance to EN 13624 and EN 14562 (high contamination): 3.0% 2 min. Enveloped viruses (i.e. HBV, HIV, HCV) in accordance to BGA (RKI)/DVG (contamination): 2.0% 2min. Safety Instruction: H 314 Causes severe skin burns and eye damage. H 411 Toxic to aquatic life with long lasting effects. Avoid release to the environment. Wear protective gloves/eye protection. IF ON SKIN: Wash with plenty of water. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If skin irritation occurs: Get medical advice/attention. If eye irritation persists: Get medical advice/attention. Dispose of contents/container in accordance with all local, regional, national and international regulations. Contains N,N-Dimethyl-N-Dioctylammoniumchlorid, Potassium hydroxide.

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2L e

DE Healthcare Products
DEDICATED TO EXCELLENCE

CB Healthcare Consulting GmbH
(manufacturer)
Am Neumarkt 34
22041 Hamburg, Germany
Tel: +49 40 656680
Rev.2019/06

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