## TEMPORARY CEMENT WITH ZINC OXIDE EUGENOL

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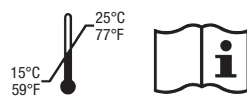
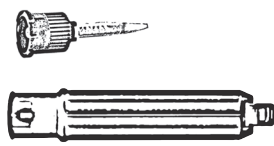
3x  
2x5ml

DEHP DE Healthcare Products  
DEDICATED TO EXCELLENCE

## ENGLISH

### INSTRUCTIONS FOR USE

DE Healthcare Temporary cement with zinc oxide eugenol is automatically mixed zinc oxide eugenol-based cement.

#### Indications

- Temporary luting of temporary and permanent crowns, bridges, inlays and onlays.
- Temporary luting of implant-borne dental prosthesis.

#### Contraindications

Do not use the material in cases of allergies to any of the components or in the event of contact allergies.

#### Side effects

In individual cases, pulp damage can occur if eugenol-containing materials are used.

#### General safety instructions

- Keep out of the reach of children!
- For dental use only!
- Avoid contact with skin and eyes! In the event of inadvertent contact, immediately rinse thoroughly with plenty of water and consult a physician if necessary.
- Contains rosin.

#### Using the double-barreled syringe

The material is mixed and dispensed by extruding through a mixing tip. When using a new cartridge, remove the transportation cap. Attach the mixing tip (DEHP 9792495) and an intraoral tip

if necessary.

On each first use with the mixing tip, extrude and discard a pea size amount to ensure a uniform mix. After use, the mixing tip acts as a cap and should be left on the cartridge. Replace the mixing tip with a new one (DEHP 9792495) when the cartridge is used again.

#### Instructions for use

Cements containing eugenol can inhibit the curing of composites, resin-based luting cements and bonds. Check whether eugenol-containing cements are suitable for use.

#### Recommended use

1. Apply the material to the dry temporary restoration.
2. Insert the temporary restoration onto the dry, prepared tooth within 1 minute using slight pressure.
3. After curing, remove excess cement with a probe or scaler.
4. Remove interdental excess carefully using dental floss.

#### Composition

Catalyst paste: Natural resins, eugenol, additives

Base paste: Zinc oxide, paraffin, additives

#### Technical data

Working time: ≥1 min (at 23 °C; incl. mixing time)

Intraoral curing time: 7 min (at 37 °C)

#### Classification

Instructions for use DE Healthcare Temporary cement with zinc oxide eugenol complies with the requirements of ISO 3107:2011, type I.

#### Storage

Store in a dry place between 15–25°C/59–77°F. Do not store in the refrigerator. Do not use after the expiration date.

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