

# CYBERTECH

## Gutta Percha Points

Gutta percha points are root canal obturation material. It is used to fill the cleaned, shaped and irrigated root canal space.

- Time saving with easy filling
- For faster and easier root canal obturation Variable tapers & sizes
- Length marking available
- Excellent radiopacity
- Long shelf life and freshness assured
- Suitable for vertical and lateral condensation

### INDICATION AND PURPOSE

1. Gutta percha point is an integral material in promoting periapical healing and preventing disease progression.
2. Endodontic therapy system using the gutta percha point obturated impermeable to liquid and bacteria

### INSTRUCTION FOR USE

1. Prepare root canal by file.
2. After clean and dry the canal with EDTA solution, apply the sealer into the root canal wall by paper points.
3. Select master gutta percha point matched in prepared root canal.
4. Irrigate a gutta percha point with 5.25% NaOCl disinfecting solution.
5. Insert mater points in to the root canal as working length.
6. Insert accessory points into the root by finger spreader.
7. Confirm a filling by the x-ray.

### CAUTIONS

1. Do not use this product for any patients with a history of hypersensitivity reactions to it, such as a rash, dermatitis, etc.
2. Some patients have an allergic reaction to Trans 1,4 Polyisoprene materials and/or even experience hypersensitivity. Do not use this product for them
3. This product is intended only for use for the tissue inside the mouth. Be careful to prevent it from coming into contact with eyes. If material has come into contact with eyes or skin, wash immediately with plenty of water and then consult a doctor.
4. Non-specialists are prohibited from using this product. Do not use this product for any purposes other than its intended purposes.
5. Do not prescribe an internal medicine
6. Should be careful to avoid overfilling or dead space
7. Be sure to take measures for the protection of the respiratory tract before
8. Always close the slide lid after use space

### STORAGE

Store at room temperature of 1°C~30°C(33°F~86°F)

### SHELF LIFE

4 years form manufacture date

### AVAILABLE SIZE

Type	Sizes
Standardized (ISO Specifications)	15, 20, 25, 30, 35, 40, 45, 50
Special Taper (.04, .06)	15, 20, 25, 30, 35, 40, 45, 50

- Gutta Percha Points are available as single size or assorted size packages.

Regular packing of a sliding type dispenser: 120 points

- Gutta Percha Taper Points are available as single size or assorted size packages.

Regular packing of a sliding type dispenser: 100 points

### Symbols



Use By



Batch Code



Authorized representative in the EC



Caution, See instruction for use



Do not reuse



Manufacturer



Date of manufacture



Keep away from sunlight



Temperature limitation



CE Mark and identification number of notified body. Product conforms to the essential requirements of the Medical Device Directive 93/42/EEC



**CB Healthcare Intl. s.r.l.**

Via Partisani, 3

47016 Fiumana, Predappio (FC), Italy

Rev.2017/03