INSTRUCTIONS FOR USE

CyberCement Resin SE

CyberResin Cement SE is permanent resin cement does not require etching, priming or bonding. Self / light cured. The "Automix" system provides a rapid, consistent and homogeneous mixture which can be dispensed directly into the restoration or tooth cavity.

INSTRUCTIONS:

- Remove the temporary restoration. Remark: Please only use temporary cements without eugenol. Eugenol-containing materials products can inhibit the polymerization process.
- Rinse carefully
- Dry the preparation with air (do not dry up).
- Try in the restoration to ensure proper fit.
- Bleed the syringe before initial use. Place mixing tip on the Automix syringe.
- Dispense the material as follows: Inlay/Onlay: Mount the supplied intra oral tip onto the mixing tip and dispense the cement directly covering all surfaces. Put the restoration gently onto the preparation allowing the material to flow from all sides.
- Crown: Dispense the cement directly into the restoration covering all surfaces. Place the restoration smoothly onto the preparation allowing the cement to flow from all margins.
- Post: Mount the supplied intra oral tip onto the mixing tip. Put the cement into the canal. Place the post and move slightly avoiding trapped air. Help the cement to flow from the canal space. When the restoration is properly placed, please remove all extra cement. Excess cement is best to be removed when it is still in gel stage with a scaler / explorer. Gel consistency can be achieved by curing excess with a light for 2 seconds only or allowing to self-cure for 1.5 - 2 minutes after application. To achieve the best results, leave it undisturbed for 1.5 minutes before light curing.

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CB Healthcare Consulting GmbH