CYBERTECH

MATRIX

- (EN) Sectional Matrix System – **INSTRUCTIONS FOR USE**
(ES) Sistema de matrices seccionales – **MODO DE EMPLEO**
(FR) Système de matrices sectionnelles – **MODE D'EMPLOI**
(DE) Teilmatrizensystem – **GEBRAUCHSANWEISUNG**
(IT) Sistema di matrici sezionali – **ISTRUZIONI PER L'USO**

Kit - 988 2151

50xS 50xM 50xL

Refills

50 x S
988 2152

50 x M
988 2153

50 x L
988 2154

988 2156
x 50

988-2155
x 2
+
8 x 

EN INSTRUCTIONS FOR USE

The Cybertech sectional matrix system, indicated for Class II restorations, is composed of preformed sectional matrices available in 3 sizes, including rings for its attachment and silicone tips. The sectional matrix provides an easier reconstruction of contact points, maintains the tooth's natural anatomy and patient comfort. The ends of the ring are equipped with silicon tips that provide a better set adaptation. **Use:** Select the appropriate matrix size for the cavity that is to be restored. Install the matrix in the proximal face that is to be restored. In the case of a MOD cavity, it is recommended one face at a time. Install the wedge. Install the ring. In concluding the restoration, first remove the ring, then the wedge and finally the matrix. To obtain consistent contact points it is recommended to promote a previous separation of the adjacent tooth, during the initial patient preparation, by inserting a wedge and/or placing the ring.

Matrix Shapes: Available in small, medium and large sizes. **Physical principle of operation:** Mechanical device to grant tooth natural anatomy to the restorative material. **Warnings, precautions and restrictions:** Professional use only. When there is proven allergy to any of its components, the product must not be used. Excessive expansion with the pliers can result in poor staple formation. During the ring insertion, hold your finger on it until removing the pliers, ensuring the correct fit and preventing from slippage. Be careful during the ring insertion, due to its flexibility. Make sure that the product is fully seated in the pliers. Avoid leaving the pliers open for a long period of time. It is recommended to protect the papilla by any method to prevent accidents. To avoid an accidental swallowing or aspiration of this product we always recommend the application of an isolation

technique of the operator field. The ring must be used with the silicone tips for a proper adjustment of the matrix. The ring and silicone tips must be sterilized separately. Sterilization may cause slight change in color, however, it does not affect its functionality. **Point of Use:** Products should be cleaned as soon as possible before and after use to prevent soils from hardening that may compromise post cleaning.

Sterilization: Steam autoclave sterilization only. Products are supplied not sterile, sterilize prior to first use. Clean using water and enzymatic detergent. The rings and silicone tips are sterilizable in autoclave (134°C, 6 min.), following the autoclave manufacturer's instructions, with a 20-30 minute drying time. Keep product sealed in the sterilization pouch until ready to use. Store the product only in CE/FDA approved sterilization pouch. The matrices are for single use only. Reuse can cause cross contamination.

Composition: Matrices: Stainless steel. Ring: Nickel plated carbon steel and thermoplastic material. Tips: Silicone.

For professional use only.

According to the EU Medical Devices Regulation, users / patients are obliged to report serious events with a medical device to the manufacturer and to the competent authority of the country in which they occurred.

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CB Healthcare Consulting GmbH