CYBERTECH

MATRIX

Sectional Matrix System INSTRUCTIONS FOR USE

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INSTRUCTIONS FOR USE

The Cybertech sectional matrix system, indicated for Class II restorations, is composed of preformed sectional matrices available in 3 sizes, including rings for its attachment and silicone tips. The sectional matrix provides an easier reconstruction of contact points, maintains the tooth's natural anatomy and patient comfort. The ends of the ring are equipped with silicon tips that provide a better set adaptation. **Use:** Isolate the operative field with a rubber dam or other suitable isolation technique. Select the appropriate matrix size for the cavity that is to be restored. Install the matrix in the proximal face that is to be restored. In the case of a MOD cavity, it is recommended one face at a time. Install the wedge. Install the ring. In concluding the restoration, first remove the ring, then the wedge and finally the matrix. To obtain consistent contact points it is

EN

recommended to promote a previous separation of the adjacent tooth, during the initial patient preparation, by inserting a wedge and/or placing the ring. **Matrix Shapes:** Available in small, medium and large sizes. **Warnings, precautions and restrictions:** Professional use only. The matrices are of single use. Their reuse can cause cross-contamination. When there is proven allergy to any of its components, the product must not be used. Excessive expansion with the pliers can result in poor staple formation. During the ring insertion, hold your finger on it until removing the pliers, ensuring the correct fit and preventing from slippage. Be careful during the ring insertion, due to its flexibility. Make sure that the product is fully seated in the pliers. Avoid leaving the pliers open for a long period of time. It is recommended to protect the papilla by any method to prevent accidents. To avoid an accidental swallowing or aspiration of this

product we always recommend the application of an isolation technique of the operatory field. The ring must be used with the silicone tips for a proper adjustment of the matrix. **Sterilization: RING / SILICONE TIPS - Cleaning:** Carry out the cleaning with the aid of water and enzymatic detergent 4 enzymes (Amylase, Protease, Lipase and Carbohydrase) and with pure pH range between 6,5 - 7,5. Prepare the solution according to the manufacturer's instructions. The water should be potable. For a better effectiveness, it is recommended that the temperature be between 40°C and 55°C. Submerge the products and leave in contact for 5 minutes with the solution. Remove parts and rinse under running water for at least 10 seconds. Dry the pieces with absorbent paper or disposable wipes. **Sterilization:** The ring and the triangular silicone tips must be sterilized separately in autoclave until 134 °C - 6 min. The sterilization can cause slight change in the

coloring of the tips and reinforcement, but does not affect its functionality. **Inspection:** Check the product before each use. Note if there is damage (breaks/cracks/rust) that prevents proper operation and does not use in such cases. These conditions are indicative of fragility of the material, whose strength can be reduced during clinical use. **Reprocessing Limits:** This is a reusable device. Repeated processing has minimal effect due to its composition. The end of the life of the ring and triangular silicone tips, identified by the professional, is determined due to damage resulting from use. **Composition:** Matrices: Stainless steel. Ring: Nickel plated carbon steel and thermoplastic material. Tips: Silicone. **For professional use only.** According to the EU Medical Devices Regulation, users / patients are obliged to report serious events with a medical device to the manufacturer and to the competent authority of the country in which they

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