# CYBERTECH

**NANOCOMPOSITE 2**

## Instructions for use

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## Mode d'emploi

## Gebrauchsinformation

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**DE**

**GEBRAUCHSINFORMATION**

**Definition**

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**INSTRUCTIONS FOR USE**

**Definition**

Cybertech Nanocomposite 2 is a highly filled, radiopaque, universal nano-hybrid composite. Cybertech Nanocomposite 2 covers a wide range of tooth shades with fewer shades.

**Composition**

Methacrylates
Barium glass, silanized
Amorphous silica, hydrophobed

**Technical Data**

Complies with ISO 4049

Range of dimensions of inorganic filler particles: 0,04-2,9 µm
Inorganic filler content by volume: 58 %
Inorganic filler content by weight: 77 %

**Indications**

- Cybtech Nanocomposite 2 is specifically indicated for:
  - Direct filling of class I, II, III, IV and V cavities
  - Luting and repairing of ceramic and composite restorations
  - more translucent material in the layer technique

**Contra-indications**

If the site cannot be isolated after enamel etching and during application and curing of Cybertech Nanocomposite 2. If allergic to components of Cybertech Nanocomposite 2. If oral hygiene is poor.

**Side effects**

No systemic side effects are known. Contact allergies with products of similar composition have been reported in isolated cases.

**Interactions with other agents**

Agents which contain eugenol and /or oil of cloves may af-fect the polymerization of Cybertech Nanocomposite 2. The use of zinc-oxide-eugenol cements in combination with Cybertech Nanocomposite 2 should therefore be avoided. Discolorations may occur when using cationic mouth rinses, plaque indicators or chlorhexidine.

**Application**

**Cleansing and isolation**

Cleanse the tooth to be treated and the adjacent teeth. Select color and create a dry working area.

**Preparing the cavity**

When preparing the cavity, every effort should be made to use a technique that is gentle to the tooth substance (prin-ciple of Adhesive Restoration Technique). Enamel edges in the anterior region are to be beveled. In the posterior region, only sharp enamel edges should be slightly beveled or rounded.

**Pulp Protection**

A suitable pulp-protective liner should be applied in close proximity to the pulp. Procedure according to manufacturer’s instructions for use.

**Application of bonding agent**

Apply the bonding agent. Procedure according to manufacturer’s instructions for use.

**Application of Cybertech Nanocomposite 2**

Dispense material out of the syringe or tip and apply into the cavity by using the increment technique.

**Polymerisation**

Cybtech Nanocomposite 2 can be cured using any light curing unit.

Maximum thickness of the layer and curing times:

		≥800 mW/cm²	≥1600 mW/cm²
	<b>Thickn-ess of the lay-er</b>	<b>exposure time</b>	
<b>Cenamel: A1, A2, A3, A3,5, A4, B2, B3, D3, C2, C3</b>	2 mm	20 s	10 s
<b>dentine: A2, A3</b>			

**Note:** Cybertech Nanocomposite 2 must be cured layer by layer. Do not remove the inhibition layer since this would affect adhesion between the layers. In the event of under-exposure there is a risk of insufficient curing. Cybertech Nanocomposite 2 is light-sensitive. Avoid conti-nuing exposure to direct sunlight, especially the operation-and/or sunlight.

**Finishing**

From coarse to fine with diamond finishers (40 µm and 15 µm) with a constant wiping motion and copious spraying with water. For optimal polishing results polishers are recommended.

**Emergency measures**

In case of direct contact with the oral mucosa, rinsing with tap water is sufficient. In the case of contact with the eyes, rinse thoroughly with water (10 min) and consult physician.

**Note**

Only supplied to dentists and dental technicians or upon their instructions. Keep out of the reach of children! Tips are for single patient use only. Recap composite sy-ringes after use.

**Shelf life and marking**

The expiry date and  number are marked on the syring-es and tips.

**Storage**

Store at 4–23 °C / 39–73 °F.

Avoid exposure to direct sunlight or other heat sources.

**Caution**

Federal law restricts this device to sale by or on the order of a dentist.

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