# MAXIMA ACCLEAN AIR POLISHING POWDER 40MM LEMON/MINT

 Before using this product, please carefully read and follow the recommendations of the instruction manuals originally delivered with your device and system. Please pay special attention to the safety precautions.

Any serious incident that has occurred in relation to the device should be reported to the manufacturer and the competent authority of the Member State in which the user and/or patient is established.

## Intended use

The Maxima Acclean Air Polishing powder 40µm Lemon/Mint is intended to be remove biofilm, stains and early calculus supragingivally above cemento-enamel junction:

- From secondary teeth.
- From sound enamel.

## Indications

This powder is indicated for air polishing: Removes biofilm supragingivally, stains and early calculus.

## Intended user population

This powder is intended to be used by dental professionals.

## Intended patient population

This powder is intended to be used in population with secondary teeth, including children.

## Undesirable side effects and contraindications

Flavors aromas (lemon or mint) can cause allergic reactions. This powder should not be used on patients allergic to flavor aromas (lemon or mint).

This powder contains salt and should not be used on patients with a low salt diet or hypertension.

## Recommendations

 Do not use this powder in the direction of oral soft tissues or any dental tissue other than sound enamel.

## Storage and shelf-life

Storage condition up to 30°C and up to 90% relative humidity.  
Transport conditions between -29°C to 38°C and up to 90% relative humidity.  
Shelf life: 36 months.

## Disposal

There is no specific safety risk related to the disposal of the product and its container. Disposal must be made according to country specific regulations.

## Compatibility

Compatibility tests have been performed with EMS AIRFLOW® devices equipped with EMS AIRFLOW® Handpiece

## Main ingredient

Sodium Bicarbonate

## Contains

Sweetener (sucralose for Mint flavor / saccharin for Lemon flavor)  
Aroma  
Hydrophobic modified amorphous silica

 Translations in this manual are based on the English version. Thus the English text will be referred to in case of discrepancy.

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**Dr. Wittmann GmbH & Co KG**