SELACRYL IMPLA

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EN: SELACRYL IMPLA is a fast setting, hot-curing High Impact resin on a methylmethacrylate basis for full and partial dentures. SELACRYL IMPLA is intended for use with the pressing technique. SELACRYL IMPLA distinguishes itself for excellent consistency, high mechanical values, natural colour reproduction and minimal shrinkage during polymerization. **Preparation:** Invest the wax-up, carefully boil out the flask and cover the plaster surfaces with separator on an alginate basis (2x). Important: when applying the second insulation layer, the model surface must not be completely dry; allow the model to dry after applying the second layer! This is extremely important to avoid the white discolouration of the base area. **Mixing:** 10 g powder : 5 g liquid (or as desired). Pour the liquid into the mixing bowl, add the powder and stir. Then mix thoroughly with a large spatula. After approximately 10 minutes, at room temperature: 18-24 °C, the paste is ready for use within 45 minutes. **Pressing:** Put the acrylic dough with excess material in the cooled flask. Slowly close the flask, proceed with a trial pressing and press again if necessary. Maintain pressure for approx. 10 minutes. Polymerization: SELACRYL IMPLA was especially developed for quick polymerization. Place the flask in boiling water (95-98°C – approx. 3 l water for a flask) and allow to polymerize for 20-30 minutes. In case of particularly large models: place the flask in warm water (50-60°C) for 15-20 minutes, heat up until it boils (95-98°C) and allow to polymerize for 15-25 minutes. SELACRYL IMPLA can also be used with other polymerization methods. **Storage and safety measures:** Material must not be used after expiration date. Storage temperature must not exceed 25 °C (77 °F). Avoid direct sunlight. Keep containers closed after use and keep out of reach of children. Monomer is highly flammable. Irritating to respiratory system and skin. Sensitization may occur after skin contact. Keep containers in a well ventilated place. Avoid eye and skin contact. Do not inhale vapors. Keep away from sources of ignition. No smoking. Do not allow to get into drains. Empty containers can be recycled in accordance to your local recycling station. **Classification according to DIN EN ISO 20795-1 Type 1 - medical device class IIa.** According to the EU Medical Devices Regulation, users / patients are obliged to report serious events with a medical device to the manufacturer and to the competent authority of the country in which they occurred. Cont. Methyl methacrylate. Highly flammable liquid and vapour. Causes skin irritation. May cause an allergic skin reaction. May cause respiratory irritation. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Avoid breathing dust/ fume/ gas/ mist/ vapours/ spray. Wear protective gloves/ protective clothing/ eye protection/ face protection. IF ON SKIN: Wash with plenty of soap and water. If skin irritation or rash occurs: Get medical advice/ attention. Take off contaminated clothing and wash it before reuse. In case of fire: Use Carbon dioxide (CO₂), foam or dry powder to extinguish. Store in a well-ventilated place. Keep cool. Dispose of contents/ container as hazardous waste - in accordance with local and national legislation - suitable, approved incinerator for combustible organic waste.

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EN: Danger
FR: Danger
DE: Gefahr
NL: Gevaar


UN1247

 **CB Healthcare Consulting GmbH**
(manufacturer)

Am Neumarkt 34
22041 Hamburg, Germany
Tel: +49 40 656680

 0482
Rev.2020/01

Medizinprodukt
Medical Device
Dispositif médical
Medisch hulpmiddel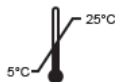

Email: info@cbhealthcare-intl.com