## ECO-Sil Gebrauchsinformation

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## ECO-Sil Instruction for use

Elastomer precision impression material on a polyvinylsiloxane basis, addition-curing.

Thank you for reposing your trust in us. With your purchase of ECO-Sil, you have made the right decision.

### Product description:

ECO-Sil is an impression material that meets highest demands. The correction materials are highly thixotropic, with excellent recovery capacity and pronounced hydrophilic properties. All materials have a thermo-activating formulation that allows variable working times, which translates into a high degree of working comfort. The time in mouth is patient-friendly because it is always uniformly short, no matter when the impression tray is inserted in the mouth. The system is complemented by a contrast-rich colour system and a pleasantly mild mint flavour.

### Indication:

- ECO-Sil putty soft: Pre-impression material for putty wash technique (2-step) and/or base material for simultaneous putty wash technique (1-step).
- ECO-Sil medium, light, light fast set: Correction material for both impression techniques

### Processing:

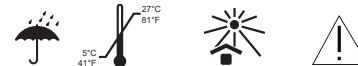
Dispense the required amount of ECO-Sil putty soft at a catalyst 8ml (ca. 14 g)/base 8ml (ca. 14 g) dosage ratio of 1:1; mix well by hand for approx. 30 seconds, press into impression tray, and insert in mouth. When total setting time is reached, remove tray from mouth. Close containers tightly after use. Insert cartridge in 1:1 mixing gun, remove cap. When using a new cartridge, press out a small amount of material until a smooth flow is achieved. Screw mixing tip (take care to keep to the identical colour coding for cap/mixing tip) onto cartridge and dispense desired amount of material. The used mixing tip remains on the cartridge. Before using again, remove the used mixing tip and check the discharge openings. Clean if necessary, then attach a new mixing tip and continue as normal.

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### Special instructions:

- Some solutions used prior to impression, such as mouthwash solutions, may interfere with the setting process; any residues must therefore be thoroughly removed. Likewise, iron-containing retraction cords have a negative influence on the setting process.
- Do not leave any residues of the impression material in the oral cavity.
- The fully automated mixing rules out the negative effects of the material through contact with certain glove types, e.g. latex gloves. This sensitivity exists with all addition-curing silicones; it must be taken into account during processing.
- Cartridge materials should be used at room temperature (64.4°F - 77°F, or 18°C - 25°C) only.
- Disinfection: Impressions can be disinfected with 2% glutaraldehyde solution.
- Galvanising: The impressions can be plated with copper or silver by galvanisation.
- Creation of models: The impression can be poured after only 60 minutes, e.g. with Type 3 or 4 plasters that meet the requirements of ISO 6873. There are no other time limits to be observed. The impressions can be poured several times.
- Storage: between 41°F and 80.6°F (5°C - 27°C) Shelf life: see information on package.

### Operating and safety instructions

Avoid contact with eyes; if necessary wash eyes immediately with plenty of water, and consult an eye specialist without delay. Avoid contact with clothes, as curing silicones are chemically stable and cause smudges that cannot be removed. The processing, usage and application of the products delivered by us fall exclusively in the domain of your responsibility, as it is not possible for us to exercise any control in this regard. To the extent that we offer oral or written advice or provide consultancy through technical application-related experiments, any such consultancy is to be regarded as non-mandatory instructions only, even with regard to possible industrial property rights of third parties. This does not release you from the obligation of testing the delivered products on your own, to ensure that they meet the intended objectives and processes. Our liability is limited in all cases and all instances of damage to the value of the products delivered by us and used by you. The faultless quality of our products is guaranteed according to the standard of our general business conditions.

**DE Healthcare Ltd**  
Unit 9, Kingsthorpe Business Centre,  
Studland Road, Kingsthorpe,  
Northampton NN2 6NE U.K.  
Rev. 2015/06 Subject to alterations. Errors excepted!

### Technical data and product properties

Technical data	ECO - Sil putty Soft	ECO - Sil medium	ECO - Sil light	ECO - Sil light fast set
Consistency / ISO 4823	Kneadable Type 0	Medium-bodied Type 2	Light-bodied Type 3	Light-bodied Type 3
Colour	Dark blue	Lilac	Pink	Pink
Dosage (1:1) / mixing time	30 sec.	automatic	automatic	automatic
Variable working time including mixing time	up to 2 min.	up to 2 min.	up to 2 min.	1 min.
Time in mouth	2 min.	2 min.	2 min.	1 min.
Total setting time	to 4 min.	to 4 min.	to 4 min.	2 min.
Deformation under pressure	2 %	2 %	3 %	3 %
Recovery from deformation	99,4 %	99,6 %	99,6 %	99,6 %
Linear dimensional change	0,2 %	0,13 %	0,16 %	0,16 %
Flavour	mint	mint	mint	mint

Time specifications given in the table relate to 73.4°F (23°C) and a relative humidity of 50+/- 10%. As a general rule, higher temperatures will result in shorter setting times, and lower temperatures in longer setting times.

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