SAFE-ST PLUS SURFACE DISINFECTION WIPES

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EN: Type of application: Ready-to-use alcohol-based wet wipes for protective surface disinfection of alcohol-resistant non-invasive medical devices. Notes on application: Open the refill pack. Place the contents, including bag, into the container. Pull the first cloth from the middle of the roll into the lid of the container and replace the lid. Please only refill the container with the correct pack of Disinfection Wipes. Carefully wipe the area to be disinfected and leave for the appropriate contact time. Only for use on alcohol-resistant materials (e.g. not suitable for acrylic glass). Once the bag has been opened and placed in the container the cloths can only be used for 28 days. The opening date, best before date as well as the relevant legal references of the liquid have to be observed and applied to the label for the external identification of the container. After taking out a cloth, close the container carefully and seal. The inside and outside of the container should be prepared using a suitable disinfectant cloth. For professional use only. **Application times [related to the disinfection liquid]:** Bacteria as per EN 13727, EN 1276, EN 13697, EN 16615 [dirty]: Conc. 1 min; Candida albicans as per EN 13624, EN 1650, EN 13697, EN 16615 [dirty]: Conc. 1 min; TB [tuberculosis] pathogens [M. terrae] as per EN 14348 [dirty]: Conc. 1 min; Mycobactericide [M. avium] as per EN 14348 [dirty]: Conc. 1 min; Enveloped viruses e.g. HBV, HCV and HIV as per EN 14476 und EN 16777 [dirty]: Conc. 1 min; Noroviruses: Conc. 1 min. [EN 14476, clean]: Conc. 2 min. [EN 16777, clean]: Adenoviruses: Conc. 1 min. [EN 14476, EN 16777, clean]; Polyoma SV 40 [RK/DVV dirty]: Conc. 2 min; limited virucidal plus [Adenoviruses, Noroviruses, Rota-viruses] as per EN 14476: Conc. 1 min. [clean]; limited virucidal plus* [Adenoviruses, Noroviruses, Rota-viruses as per EN 16777: Conc. 2 min. [clean]. Application times related to the impregnated cloths: Bacteria and candida albicans as per EN 16615 [high contamination]: Conc. 5 min. **Contents:** 120 cloths made of PP/VIS blended fabric, 12 cm x 20 cm; 100 g of fluid contains: 20 g 1-propanol; 5 g 2-propanol. **Safety guidelines:** Flammable solid. Causes serious eye irritation. May cause drowsiness or dizziness. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Wear eye protection/ face protection. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. If eye irritation persists: Get medical advice/ attention. Store in a well-ventilated place. Store in a closed container. Dispose of contents/ container in accordance with local/ regional/ national/ international regulations. Flash point as per DIN 51755: approx. 29°C. Application max. 50 ml per m². The total applied amount per room must not exceed more than 100 ml per m² of room area. Keep the containers tightly sealed. Place the packaging into the recycling system ONLY if it is empty of all residue. Dispose of residual product in accordance with EAK 07 D6 99.

(01)04062183011897

see imprint

LOT

120

0 °C
32 °F

40 °C
104 °F

MD

Medical device / Dispositivo médico
Dispositiv médical / Medisch hulpmiddel
Dispositivo médico

CB Healthcare Consulting GmbH

Am Neumarkt 34
22041 Hamburg, Germany
Tel: +49 40 656680

DE Healthcare Products
DEDICATED TO EXCELLENCE

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