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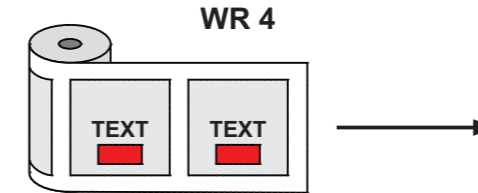
EXP:

1L e BAuA Reg.-Nr.: N-86478

DE Healthcare Products
DEDICATED TO EXCELLENCE

CB Healthcare Consulting GmbH
Am Neumarkt 34
22041 Hamburg, Germany
Tel: +49 40 656680
Rev. 2020/01

5°C 25°C

(EN) Rapid surface disinfection

Areas of application: Acc. to EU Medical Devices Regulation: Disinfection and cleaning of non-invasive alcohol-resistant medical devices. Acc. to Biocidal Products Regulation (BPR): quick disinfection and cleaning of alcohol-resistant surfaces of all kinds.

Spectrum of application and exposure time: Surface disinfection (wipe disinfection), bactericide, yeasticidal as per VAH/EN16615: 5 min.; tuberculocide (M. terrae): 30 sec.; active against enveloped viruses (including HBV, HIV, HCV): 30 sec., limited spectrum virucidal activity: 3 min., rotaviruses: 30 sec.; noroviruses (MNV): 1 min.; adeno viruses: 3 min.

Application: To ensure complete wetting apply the undiluted solution evenly on alcohol-resistant surfaces and keep them moist for the required contact time.
Composition: 100 g ready to use solution contains: 45 g ethanol (CAS: 64-17-5).

Warning: Flammable liquid and vapour. Keep away from heat, hot surfaces, sparks, open flames and other ignition sources. No smoking. Keep container tightly closed. Do not breathe vapour. Use only outdoors or in a wellventilated area. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Dispose of contents/container to approved disposal company or local collection.
For Professional Use Only.

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