I am type 2.

Zirlux Temp

PMMA Blank
PMMA Disco
Disque PMMA
PMMA Fräsrohling
Disco PMMA
PMMA Schijven
Disco PMMA
PMMA tarcze
PMMA Blank

Ø 15 mm

9793087	PMMA Blank with step B1
9793088	PMMA Blank with step A3
9793089	PMMA Blank with step A2

Ø 20 mm

9793328	PMMA Blank with step A2
9793329	PMMA Blank with step A3
9793330	PMMA Blank with step B1

www.zirlux.com

   #iamtypez

(manufacturer)

HENRY SCHEIN INC.
135 DURYEA ROAD
Melville, NY 11747 USA

0482

EC REP

HENRY SCHEIN SERVICES GmbH
MONZASTRASSE 2A
D-63225 LANGEN, GERMANY
WWW.HENRYSCHEINBRAND.COM

Rev.2020/10

INSTRUCTIONS FOR USE

Indications	Zirlux Temp PMMA blanks consist of a millable PMMA material. They can be used to form crowns and bridges as well as long-term temporaries. The material can be used with or without a veneer. Moreover, Zirlux Temp PMMA blanks make it possible to check fit and functionality of the dental work on a plaster model. The constructions are modeled as usual with CAD / CAM and milled from Zirlux Temp PMMA blanks.
Frame	<ul style="list-style-type: none">• A maximum of 2 pontics and max. 1 attachment can be positioned for the lateral teeth.• The minimum thicknesses (without veneer) are: 1.0 mm (occlusal), 0.6 mm (cervical)• Connector cross sections: in the incisor area >10 mm², in the lateral area > 15 mm²
Finishing / Polishing	<ul style="list-style-type: none">• Shaping is done with cross cut carbide cutters• Polishing is done with normal commercial high gloss polishing pastes and goatshair brushes
Important Milling	<p>The plastic material must not become overheated when using cutters, steam jet, etc.</p> <ul style="list-style-type: none">• Always mill the material in an ambient temperature of 20 °C– 25 °C (68 °F – 77 °F).• Please adhere to the instructions of your milling system.• Zirlux Temp PMMA Blanks are suited to dry processing only.• All tools used must be suited to processing PMMA. We recommend sharp-edged radius mills.• The milling parameters strongly depend on the milling system, on the CAM software and on the milling geometries. <p>If you are uncertain, please use your parameters for PMMA.</p>
Veneer	The veneer is made with veneer composites. A bonding agent must be used. Frame thicknesses may not be less than 0.4 mm.
Mounting	The bridge can be fixed with all temporary cements.
Lifespan	The wearing life is limited to 12 months maximum.
Storage	Store in dry place.
Side-effects	With proper use of this medical device, unwanted side-effects are extremely rare. Reactions of the immune system (allergies) or local discomfort, however, cannot be ruled out completely. Should you learn about unwanted side-effects – even if it is doubtful that the side-effect has been caused by our product – please kindly contact us.
Contra-indications/ interactions	If a patient has known allergies against or hypersensitivities towards a component of this product, we recommend not to use it or to do so only under strict medical supervision. The dentist should consider known interactions and crossreactions of the product with other materials already in the patient's mouth before using the product.
Note	Please supply the dentist with the above information, if this medical device is used to produce a special model. Please also consider the safety data sheets. According to the EU Medical Devices Regulation, users / patients are obliged to report serious events with a medical device to the manufacturer and to the competent authority of the country in which they occurred.

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