## INDICATIONS:

For the preventive sealing of pits and fissures.

## PROPERTIES:

- Easy & precise application by using direct syringe technique.
- Light-curing.
- Low viscosity, excellent flow properties, optimal wetting of fissures.
- Fluoride release.

1. Clean the enamel surface thoroughly with either cleansing paste containing neither oil nor fluoride, or with pumice and brush

3. Apply the etching gel, and let it react for 30 to 60 seconds.

5. Dry with air free of water and oil. The etched enamel should have a mat white appearance. Avoid contamination of the etched surface with saliva.

7. Cure sealant for 20 sec.

8. Check occlusion.

2. Isolate the working field, preferably with a rubber dam.

4. Rinse thoroughly.

6. Mount and secure lock a new dispensing tip on the Henry Schein Pit & Fissure Sealant syringe and carefully syringe an adequate amount of Henry Schein Pit & Fissure Sealant into the fissures. Let the sealant flow until the entire fissure is covered. You can use a disposable brush, micro-brush or the dispensing tip itself to spread the sealant and eliminate air bubbles.

EN

## CAUTION!

Henry Schein Pit & Fissure Sealant contains polymerizable monomers, which may cause allergic contact dermatitis in susceptible persons. Wash thoroughly with soap and water after contact. If skin sensitization occurs, or if known allergy to methacrylate resin exists, discontinue use.

## ESSENTIAL INGREDIENTS

- Urethan dimethacrylate.
- Multifunctional monomers.
- Triethyleneglycoldimethacrylate.
- 4-Methacryloxyethyltrimellitic anhydride.
- Photoinitiators.
- Fumed Silica.

## STORAGE

- Do not store above 24°C (75°F).
- Keep away from direct sunlight.
- For optimum freshness, keep refrigerated.
- Do not freeze.
- If refrigerated, allow the syringe to reach room temperature.
- To obtain SDS visit [www.henryscheinbrand.com](http://www.henryscheinbrand.com)
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.

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**HENRY SCHEIN SERVICES GmbH**