ETCHGEL FOR CERAMICS

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(IT) Etching gel materiali in ceramica

Application:

1. Apply Ceramic Etching Gel HF ETCH with a small disposable brush or plastic stick to the ceramic area to be etched.
2. An etching duration of at least 60 sec. is recommended.
3. Completely remove the acid residues after the application time. Then apply an intensive steam jet to the surface, spray it with water or clean it in an ultrasonic bath with distilled water. The etching of ceramic surfaces should preferably be performed immediately before the insertion or after the try-in. Avoid contamination of the etched areas (do not brush off). However, should contamination still occur after the etching, the area must be etched again briefly (approx. 10 sec.). Dry well after the etching, then continue with the application of the bonding agent etc. in accordance with the manufacturer's instructions for use. Only for extraoral use! Ceramic Etching Gel HF ETCH must not be used for etching tooth substance in the mouth!

Fatal in contact with skin. Toxic if inhaled. Causes severe skin burns and eye damage. Avoid breathing dust/fume/gas/mist/vapours/spray. Wear protective gloves/protective clothing/eye protection/face protection. Do not get in eyes, on skin, or on clothing. IF SWALLOWED: Immediately call a POISON CENTER or doctor/physician. IF IN EYES: Rinse cautiously with water for several minutes. Remove contact lenses, if present and easy to do. Continue rinsing. Remove/Take off immediately all contaminated clothing. IF ON SKIN: Gently wash with plenty of soap and water. Specific treatment (see on this label). Call a POISON CENTER or doctor/physician if you feel unwell. Dispose of contents/container in accordance with local/regional/national/international regulations.

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HENRY SCHEIN SERVICES GmbH