## INSTRUCTIONS FOR USE

### Description:

ISO 4823 Type 3 Light-bodied Consistency - ADA 19 Type 1 Low Viscosity. ISO 4823 Type 2 Medium-bodied Consistency - ADA 19 Type 1 Medium Viscosity. CyberSilicone Vinylpolysiloxane (Hydro) is a hydrophilic addition reaction silicone impression material dispensed in an automix cartridge delivery system. CyberSilicone Hydro Impression Material is thixotropic - it resists slumping, yet flows under pressure. CyberSilicone Hydro Light Body is available in two setting types: Fast Set and Regular Set. Light Body, Fast Set is designed primarily for use with 1 or 2 unit impressions where a faster setting time is required.

**Indications:** Correction impression material for the two-step (puttywash) impression technique. Use with dispenser to extrude. Use Intra- Oral Tips for extruding material directly into mouth.

**Impression Procedure:** Inject automix material evenly around prepared teeth and/or directly onto Putty Impression. Seat tray and maintain in position for approximately 2,30 minutes (Fast Set) or 3,30 minutes (Regular Set). **Note:** Set time is based on 23°C. Higher temperature decreases set time, lower temperature increases set time.

**Casting:** Store impression at room temperature. Impression may be poured 30 minutes after removal from mouth.

**Electroplating:** Impressions may be copper or silverplated.

**Storage:** Do not store above 27°C (80°F). Shelf life 3 years when stored at 23°C (73°F).

**General Notes:** Do not use with condensation reaction silicone impression materials.

TECHNISCHE DATEN	Regular Set	Fast Set
Mixing Ratio Base : Catalyst	1 : 1	1 : 1
Mixing Time	Auto Mixing	Auto Mixing
Total Working Time* (incl. Mixing Time)	2 min.	1 min. 30 secs.
Time in mouth* (minimum)	3 min. 30 secs.	2 min. 30 secs.
Setting Time* (includes Mixing time)	5 min. 30 secs.	4 min.
Linear Dimensional Change after 24 Hrs.	<-0,20%	<-0,20%
Maximum Strain in Compression	3%-5%	3%-5%
Recovery from Deformation	>99,5%	>99,5%
Hardness « Shore A »	45±2	45±2

\*23°C/73°F

**Disinfection:** Impression may be disinfected or sterilized with a disinfectant for impressions.

**Compatibility with Gypsum:** This product is compatible with all well known gypsum materials.

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CB Healthcare Consulting GmbH