## INSTRUCTIONS FOR USE

*CyberPorcelain Fix is a complete kit for preparing porcelain veneers, inlays and fractured porcelain teeth for bonding. It consists of CyberPorcelain Etch and CyberPorcelain Silane. CyberPorcelain Etch is a specially buffered, viscous, hydrofluoric acid that will etch porcelains of all type to produce a microscopic porous surface, which will provide a strong mechanical interlock with composite resin in materials. CyberPorcelain Silane is a single component silane that enhances bonding and durability when applied to etched porcelain.*

- Preparation of porcelain crowns, veneers and inlays prior to cementation.
- Preparation of fractured porcelain or porcelain fused to metal crowns for bonding.

### INSTRUCTIONS FOR USE:

- In case of intraoral procedure use a rubber dam for isolation.
- When preparing fractured porcelain crowns for bonding, abrade the surface of the porcelain to be etched with a medium grit bur to remove the glazed surface and to make space for a sufficient layer of composite resin, for strength and aesthetics. Rinse all debris from the porcelain surface with a big amount of water and dry with moisture and oil free compressed air.
- Syringe the CyberPorcelain Etch onto the porcelain surface. Leave undisturbed for 60 seconds.
- Rinse the porcelain thoroughly with water to completely remove the etchant and dry well. The etched porcelain should have the frosted white appearance, if not, repeat the etching procedure.
- Syringe the CyberPorcelain Silane onto the etched porcelain surface, use the brush tip to spread the material. Dry well with a stream of moisture and oil free compressed air.

### ESSENTIAL INGREDIENTS

#### **CyberPorcelain Silane:**

- Ethanol
- Glycidoxypropyltrimethoxysilane

#### **CyberPorcelain Etch:**

- Hydrofluoric acid 9%
- Thickening agent
- Water

### CAUTION

CyberPorcelain Etch must be used carefully and cautiously. Avoid contact with oral soft tissues, skin and eyes. If accidental contact occurs, flush with copious amounts of water and seek medical attention immediately.

### STORAGE:

- Do not store above 24°C (75°F).
- For optimum freshness, keep refrigerated. • Do not freeze.
- Tightly recap after use.
- Do not use after expiration date. See expiry date on the label.
- Shelf life is 2 years after the date of manufacturing.

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CB Healthcare Consulting GmbH