BARBED BROACHES

• Instructions for Use

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INSTRUCTIONS FOR USE

Warning

1. Products are delivered onto the market in sterile packaging.
2. Before using instruments, check the label for the indication STERILE, the date of EXPIRATION and visually check the integrity of the package. If the package is damaged, return the package to the producer.
3. Attention, endodontic instruments are sharp.
4. Tools used for root canals can cause a puncture during disposal - when sorting waste the package must be stored in a safe rigid plastic container or containers safeguarded against opening.
5. Only dentists may work with the instruments.
6. The producer is not liable for any damage caused to the health of the patient or property.
7. Instruments contain nickel, which may cause an allergic reaction in patients sensitive to this metal.

Restriction of repeated processing: Instruments Barbed Broach are designated for single use. The producer does not recommend repeated use of the instrument due to the possibility of contamination or the transfer of infection. Consequent cleaning of the instrument is not sufficient to remove any possible contamination of the product.

INSTRUCTIONS

Storage and transport:

Store products in individual packages (blisters) in places where there is no risk of damage to the covering foil or paper. The best storage method is in group packages in paper folders. Handle the products so as to prevent damage to the package to ensure the sterility of the product. If during handling the package is damaged, return the package to the producer. Monitor the EXPIRATION date.

Checking and testing the function:

Before use, carefully visually check the product and separate all damaged, bent, broken or imperfect instruments.

Storage:

The stored products must be visibly and properly marked to prevent their improper use. The producer recommends storing the products in paper boxes, in group packages. Products in sterile packages must be stored so as to prevent package breakages, in particular damage to the covering foil or paper. When issuing from the stock, products must be taken first of all which have been stored for the longest time.

Monitor the EXPIRATION dates.

During the storage of products, it is forbidden:

- to expose products to direct solar radiation, influence of the atmosphere, thermal radiation and mechanical damage
- to use a storage place with a temperature less than -10°C or more than 40°C
- to store products in an open area

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