# SAFE-ST IMPRESSION CONCENTRATE

97-92534

Impression Material Disinfection Concentrate for Disinfection of Dental Impression Materials.

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**Instructions for use:** Dilute the concentrate with warm water [at least drinking water quality] of 10°C to 30°C (use a dispensing bottle). Place impression in the prepared solution. Ensure impression is completely submerged. Especially with Alginate and hydrocolloids, the reaction times must be observed [after the end of the reaction time take these immediately]. Remove the impression after the recommended disinfection period and rinse thoroughly with water. Do not mix with detergents or other disinfectants. The working solution can be disposed of with normal waste water. Dosage quantities and application times should be observed as precisely as possible and not be undercut. **Application times:** Bacteria as per EN 13727, EN 14561 and VAH with high bio load: 2,0% 15min. *Candida albicans* as per EN 13624, EN 14562 and VAH with high bio load: 2,0% 15min. Membrane covered viruses [e.g. HBV, HCV and HIV] as per DW/ RK with high bio load: 2,0% 15min. TbB [M. terrae] as per EN 14348 and EN 14563, low bio load: 4,0% 15 min., 2,0% 60 min. **Safety guidelines:** H 302 Harmful if swallowed. H 314 Causes severe skin burns and eye damage. H 410 Very toxic to aquatic life with long lasting effects. Avoid release to the environment. Wear protective gloves/eye protection. **IN CASE OF CONTACT WITH THE SKIN** [or hair]: Remove all contaminated clothing immediately. Wash/shower the skin with water. **IN THE EVENT OF CONTACT WITH THE EYES:** Rinse carefully for a long time with water. Remove any contact lenses if possible. Continue to rinse. Call a POISON CENTER or a doctor/physician immediately. Dispose of the contents/container in accordance with local/regional/national/international regulations. **Contains** didecyl-dimethylammonium chloride, N-(3-aminopropyl)-N-dodecylpropane-1,3-diamin. **Contents as per 648/2004/ EC:** 5-15% non-ionic surfactants, < 5 cationic surfactants, < 5% EDTA salts, fragrance.

LOT

Medical Device/Dispositivo médico/Dispositiv  
médical/Medizinprodukt/Dispositivo medicale

1 Le

 **CB Healthcare Consulting GmbH**  
(manufacturer)  
Am Neumarkt 34

22041 Hamburg, Germany  
Tel: +49 40 656680

 **DE Healthcare Products**  
DEDICATED TO EXCELLENCE

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Danger  
Peligro  
Danger  
Gefahr  
Pericolo  
Gevaar

**DEFF™ DE Healthcare Products**  
DEDICATED TO EXCELLENCE

(01)04059417016407