MAXIMA ETCH GEL 35%

Instructions for use

GEL PARA GRABADO	Instrucciones de uso
GEL DE MORDANÇAGE	Mode d'emploi
HS-ÄTZGEL	Gebrauchsanweisung
GEL MORDENZANTE	Istruzioni per l'uso
ETSGEL	Gebruiksaanwijzing

35% MAXIMA ETCH GEL

UK

WARNINGS

DESCRIPTION

35% phosphoric acid etch has been specifically designed to provide an optimum level of viscosity. It is self-limiting in its depths of etch (average depth of 1.9 µm with 20 second etch). It will flow into precise locations using a brush tip or micro tip (even into occlusal grooves for sealant

preparation). The etchant is just viscous enough to prevent migration caused by gravity and also maintain a thick enough layer to prevent premature drying. The etchant contains no glycerine and is designed for rapid and complete removal upon rinsing.

INDICATIONS

The ideal viscosity combined with the versatility of the brush or tip allow placement of the etchant in any location. The etchant is suitable for etching before bonding composites, sealants or bonding adhesives. The tips are disposable, helping to prevent cross-contamination.

PRELIMINARY PROCEDURE

1. If using preloaded 1.2ml syringes, proceed as follows:

- (a) Remove luer lock cap from syringe.
 - (b) Twist a disposable metalor brush tip as required securely onto the syringe 1.2ml.
 - (c) Verify etchant flow prior to applying intraorally
2. If using 60ml refill syringe proceed as follows
 - (a) Remove Luer cap from 60ml syringe
 - (b) Attach securely an empty 1.2ml syringe via the male thread to the 60ml syringe
 - (c) Depress the 60ml plunger while guiding the 1.2ml plunger to the desired fill level
 - (d) Separate and recap the 60ml syringe with the luer cap. Twist a disposable metal or brush tip as required securely onto the 1.2ml syringe.

PROCEDURE

1. Isolate adjacent teeth and restorations. To assure protection, a rubber dam is recommended.

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2. If enamel being etched is not freshly cut, use pumice, disc or diamond bur on the area.
 3. Make sure areas to be etched are thoroughly rinsed, cleaned and dried.
 4. Verify flow of etch before intraoral application. Dispense a small portion of gel onto a pad to familiarise yourself with its viscosity. For optimal control use the palm grasp.
- Place the tip against the surface to be etched. Slowly dispense the viscous solution while the tip is in motion. With a little practice one can lay a fine line or paint a thin layer of viscous etch limiting it to the areas being etched. This conserves gel and prevents acid from being spread unnecessarily.

5. The etchant should be applied to the area for 15-20 seconds. Rinse thoroughly, dry and proceed per adhesive manufacturer's instructions. Ensure that the etched surface is not contaminated with saliva, water or anything else so as to optimise maximum penetration of bonding system into dentine/enamel surface.

PRECAUTIONS

1. Experiment with the application of 35% Maxima Etch etch on a mixing pad or glass slab, BEFORE use on patients to become familiar with the ease with which the viscous solution moves from the tip. Never use intra-orally before checking flow from tip. Never use force on plunger to start flow intra-orally as this could result in a subsequent uncontrolled discharge of materials.

Distributed by (in US only)

Henry Schein Inc.
135 DURYEA ROAD
Melville, NY 11747 USA

Henry Schein U.K. Holdings Ltd.
Medcare House, Centurion Close
Gillingham Business Park
Gillingham, ME8 0SB U.K.
www.henryscheinbrand.com
Rev.2015/01

If resistance to flow is experienced, DO NOT proceed. Replace with new tip and verify flow before use on patient.

2. Remember that any strong acid such as phosphoric acid can burn and irritate tissues. Be careful not to splash into the eyes, onto mucous membranes, skin or any other areas. Confine to areas to be etched only. Any other area of exposure should be immediately rinsed with copious amounts of water

35% Maxima Etch gel should be stored at room temperature.

This product is not for resale by dentist.

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